

Latex-Free

Bili-Bonnet



Stretches open to allow Bili-light to penetrate the entire head.
3 Sizes Fit all head shapes.

- No Confusion
- Easy to apply
- No straps to frustrate
- No adhesives to damage skin

- Compression Technology
- Makes ours the only mask with minimal light penetration
- Rests on ocular face bones and removes pressure from eye
- Mask is Reversible made of PE Foam & Brushed Nylon
- Conforms to baby's face with plenty of room for adjustment on each side




Tabs are rounded & very low profile hook & loop that allow attachment at any angle to maximize fit and minimize slippage.

Order #	Product Description	Bx Qty
Bili-Bonnet Phototherapy Masks		
01A	Preemie size for infants approx. 27-30 weeks	50
01B	Newborn size for infants above 31 weeks approx.	50
01C	Ultra preemie size for infants below 27 weeks approx.	50

Bili-Bonnet

Latex Free-No adhesives

How to put the patented Bili-Bonnet phototherapy mask on correctly

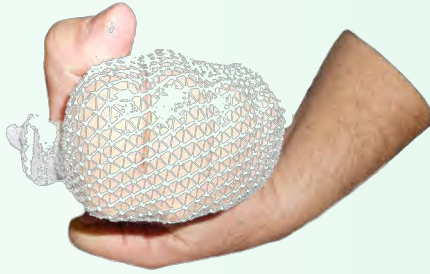

Remove from package, then stretch gauze and pull over fist to create a bonnet.



Stretch bonnet over baby's head to go from the nape of the neck to the forehead.



Attach velcro tab to one side. Note, the mask has a 1 inch area for better adjustment & velcro tabs are rounded.



Repeat with tab on other side. The mask should be snug & secure but not tight.



Ensure the netting is opened up all the way over the head to allow light to penetrate.



Now once again stretch netting down to the nape of the neck and up to the forehead.



Most Important Step
Once all is in place and secure, unhook and reattach the velcro tabs to pull up/down/midline as head shape dictates.



The end result is a mask that stays on better giving the nurse more time for other things. Protects the eyes better & allows more light to the head.

Take time to save time !