Made in the USA

MEASURING TAPE

DESIGNED TO DETERMINE APPLICATION SIZE OF

SPANDAGE TUBULAR ELASTIC SUPPORT BANDAGE

Order #

PKG

Description

Suggested Application

SAGTAPE 12/Case

SpandaGrip™ MeasuringTape measure from 6" to 43" and/or 15cm to 109cm Color Coded by Size Measure around the widest part of the area to be treated with Low, Medium or High Compression
Always apply Spanda Grip™
as a SINGLE LAYER!







How to Clean the Measuring Tape:

For sanitary reasons, MTIC recommends using the measuring tape on a single patient. If your protocol indicates that it can be used on multiple patients, the tape measure can be sanitized by wiping the entire length of the tape (both sides) with an alcohol prep pad. Let it dry completely before storing it and reusing

Spanda $Grip^{TM}$ has been designed for tissue support and provides sustained pressure. It does not provide graduated therapeutic compression. The application of Spanda $Grip^{TM}$ is solely at the discretion of the treating physician or other certified clinical personnel and should not be utilized without considering the clinical presentation of the patient and the facility's protocols and guidelines. Any information, sizing charts or other sizing instruments provided act merely as guidelines to assist in the proper application of Spanda $Grip^{TM}$, not as recommended usage for any patient or clinical condition.

